




How to send data from Contour Diabetes App

Step 1

Open Contour Diabetes App and choose menu **MY CARE**. From there, choose **REPORTS**.

Then select **RAW DATA FILE(CSV)**

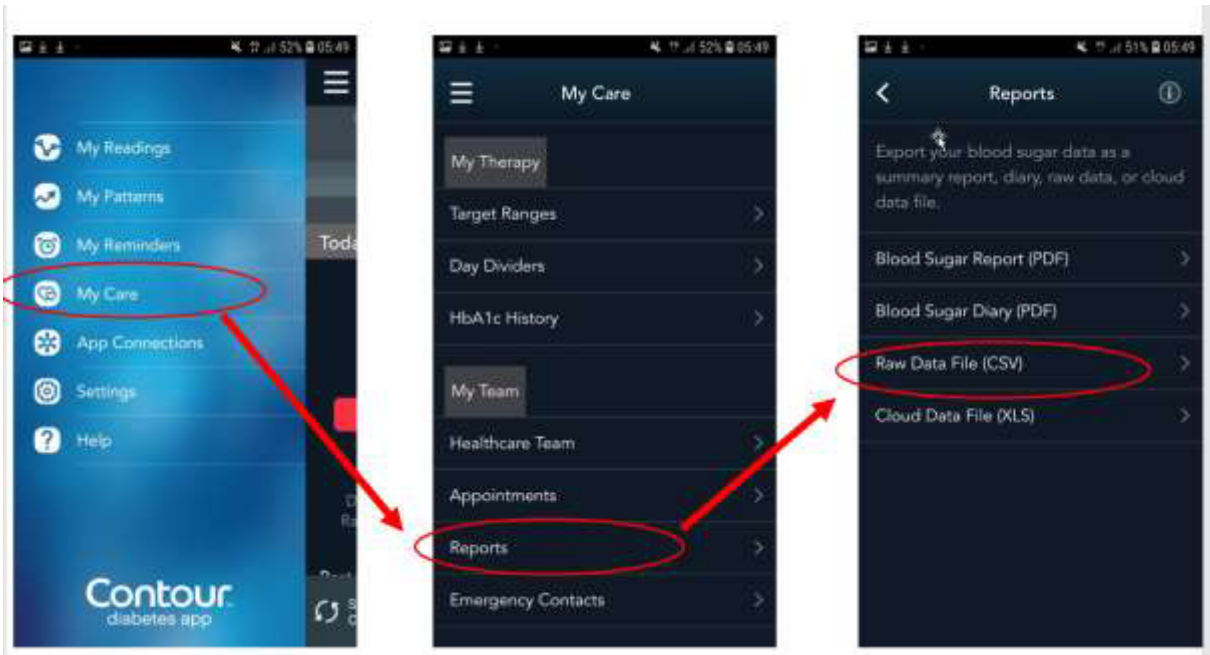


Step 2

Click on the **SHARE** symbol, located on the left bottom side of display.



In the appearing list, please choose the **DIABASS SecureSend** app.

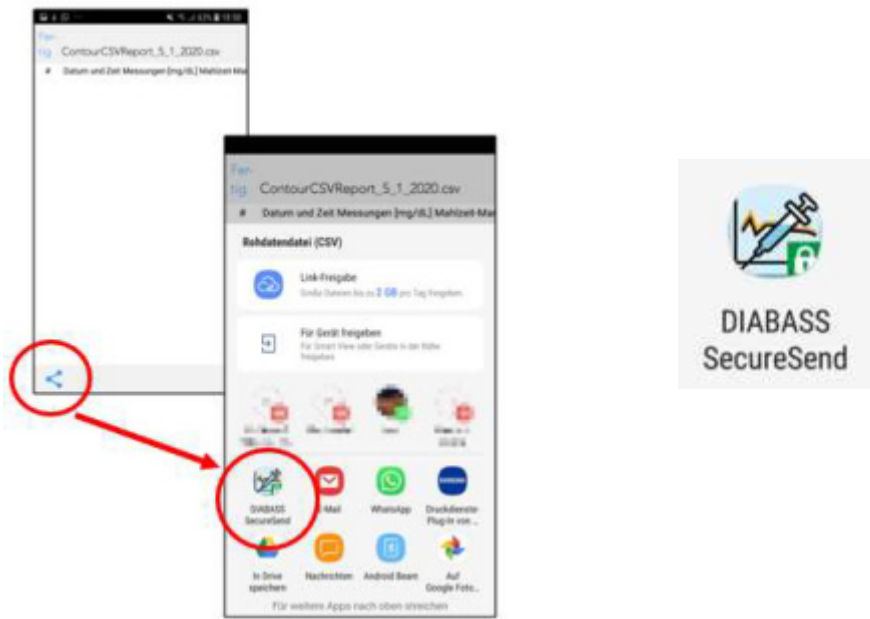




How to send data from Contour Diabetes App

Step 3

The *DIABASS SecureSend* App will now start automatically and display the file to be sent.

Please confirm with click on **NEXT**

Step 4

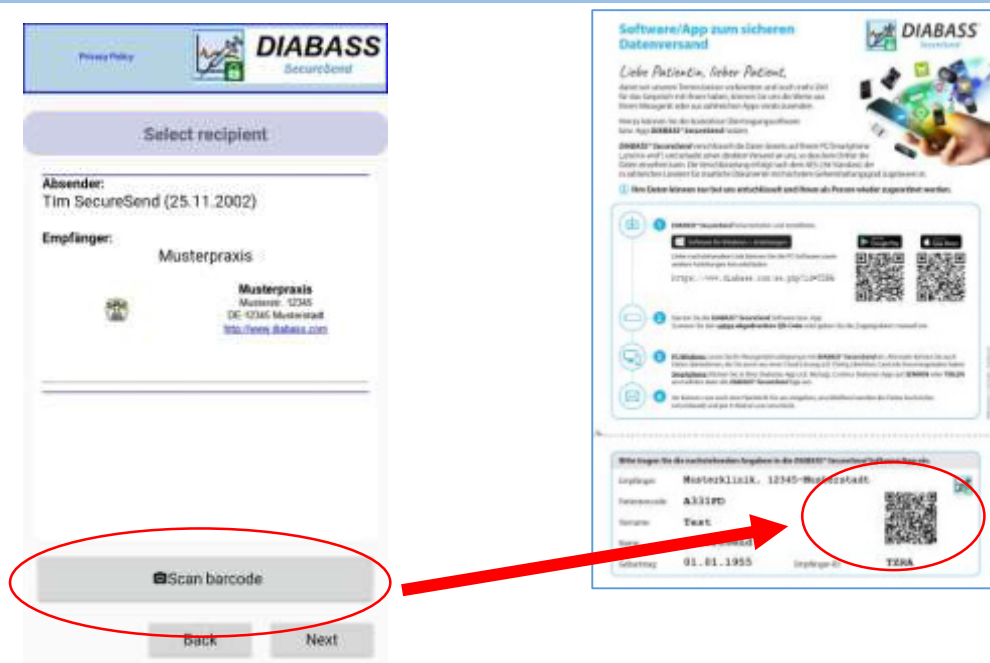

Click **SCAN BARCODE** and scan the barcode from the info sheet you received from your practice.

The recipient's information will be displayed.

Please confirm with click on **NEXT**

Step 5

In the next step, you can enter an (optional) message for the recipient.

Please click on **NEXT** to proceed.



Step 6

After some instants, the data are now highly securely encrypted (AES-256) and ready for transmission.

You can now choose how to send the data.

Sending via Email App:

An email message is generated in your default email app.

You need to send this E-Mail manually from this email app.

Sending via Server:

The encrypted data will be posted to the *DIABASS SecureSend* server. From there, the data will be forwarded to the recipient via email, using a neutral sender adress.

As soon as completed, the data will be deleted from *DIABASS SecureSend* server.

Please click on **NEXT** to proceed.

Privacy Policy

DIABASS
SecureSend

Choose data transfer

Please select transfer method

☒ **Sending via the email app**
An email is generated in the mail app which you can then send independently.

☐ **Sending via server**
The encrypted data is sent via the DIABASS SecureSend server.

Cancel Next